
**LEG ATTACHMENT BLOCK 1D60 - 1D61 - 1D63 - 1D64**  
INSTRUCTIONS FOR USE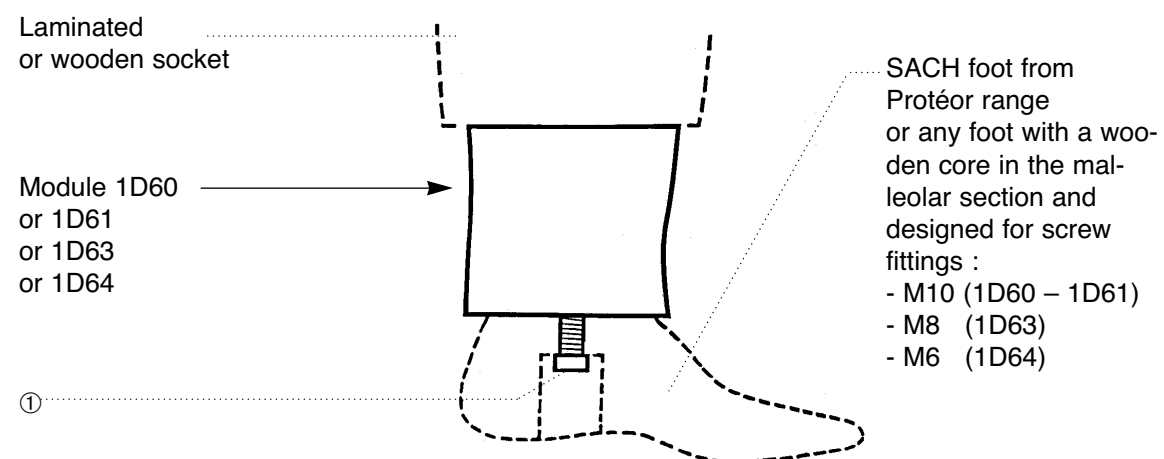

**A :** Mount the socket onto the leg attachment block as usual.

**B :** Fit screw ① into the foot and check that it protrudes from the foot by at least :  
43 mm for 1D60 – 1D61 – 1D63  
33 mm for 1D64  
then remove the screw.

**C :** Assemble the foot with the leg attachment block and direct it as required. Mark the malleolar section outline on the lower face of the leg attachment block. Remove the foot.

**D :** Screw on a wooden screw with  $\varnothing$  3 through 4 mm at the upper face of the foot wooden core. Then cut the head of this screw at about 5 mm from the upper face of the foot.

**E :** Assemble the foot again, taking care that its malleolar section corresponds to the outline drawn above **C**. Tighten strongly screw ①, so that the wooden screw cut above **D** be well marked in the wooden leg attachment block.

**F :** Remove the foot and finish the “leg attachment block/socket” assembly as usual.

**G :** At final assembly we recommend a 11N·m tightening torque value for screw ① (for the four models of leg attachment block)

**USE :**

Geriatric or slow-walking patients. Sedentary patients. Low activity level.  
Active patients involved in sports such as golf, riding, cycling, quick walking.  
Patients up to :  
100 kg for 1D60 and 1D61  
80 kg for 1D63  
60 kg for 1D64

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