GENOU 4 AXES 1M102

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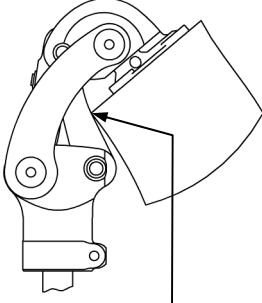
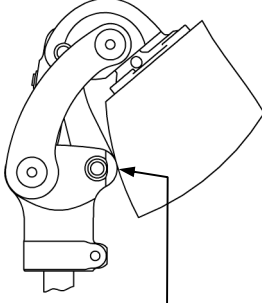
Instructions for use

A) USE

This knee joint is designed for moderately active patients whose weight added to any load being carried does not exceed 100 kg.

B) CONNECTIONS (Fig.1)

This knee can be assembled proximally (1) to any link piece for PROTEOR standard knee (1K03, 1K30, 1K160,...) and distally (2) to any Ø30 mm tube for prosthesis (1G01,...).

C) ALIGNMENT (Fig.1)

We recommend the alignment shown opposite.

D) EXTENSION ASSIST ADJUSTMENT (Fig.2)

Tighten or loosen screw 5 with a flat blade screwdriver to achieve the desired extension force.

E) EXTENSION BEARING ADJUSTMENT (Fig.2)

Caution : This adjustment affects the knee geometry and therefore the alignment. Undoing screw 6 improves flexion safety, but makes knee flexion harder. Make that adjustment with a 3 mm hex wrench when the patient weight is applied to the prosthesis in order to keep the bearing correctly in its housing.

F) JOINT FRICTION ADJUSTMENT (Fig.2 and 2-1)

Adjust only the two screws of the **right side** of the knee. Any adjustment made on the left side of the knee could damage the knee function.

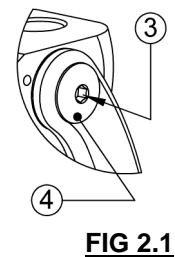
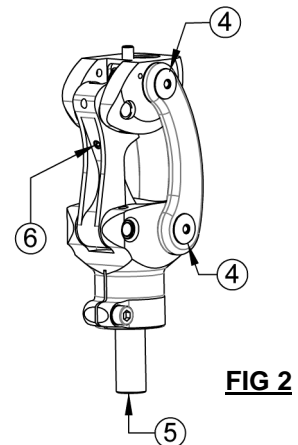
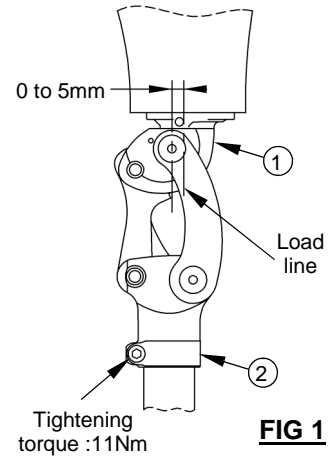
- a) Unlock screw 3 with a 2 mm hex wrench.
- b) Screw in or out the main screw 4 with a 4 mm hex wrench to achieve the desired adjustment
- c) Then lock screw 3 with a 2 mm hex wrench (tightening torque 2,5 N.m)

H) RECOMMENDATIONS

To eliminate noise in the cosmetic foam cover, do not use talcum, because it could damage the knee function and reduce the product life. Use preferably a silicone spray.

G) MAINTENANCE

Only the extension assist (ref : 1M10270) can be replaced.

CAUTION

In maximum flexion the socket must necessarily get in contact with the tube-holder.

Any other contact area is forbidden.

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