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FABRICATION

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2H200-G / 2H200-D Assembling instructions


MANUFACTURE

The complete 2H200 kit allows you to manufacture an orthosis for knee osteoarthritis. The 2H21099 assembling instructions explain how to manufacture the bands and to form the uprights using the included 2H210 kit. Please read it before starting lining.


LINING

The components of the orthosis must be separately covered with textile **before final riveting**.

1. **Cover the outside of the thigh band with adhesive fabric.** A 5 mm turn in is enough. Proceed the same way with the assembly «uprights + BK band». If the uprights will be inserted, they do not need to be covered up to their top because they will be covered by the thigh band. Pay attention to the limit drawn on the uprights because fabric must not be applied near the joint. Mark the riveting holes.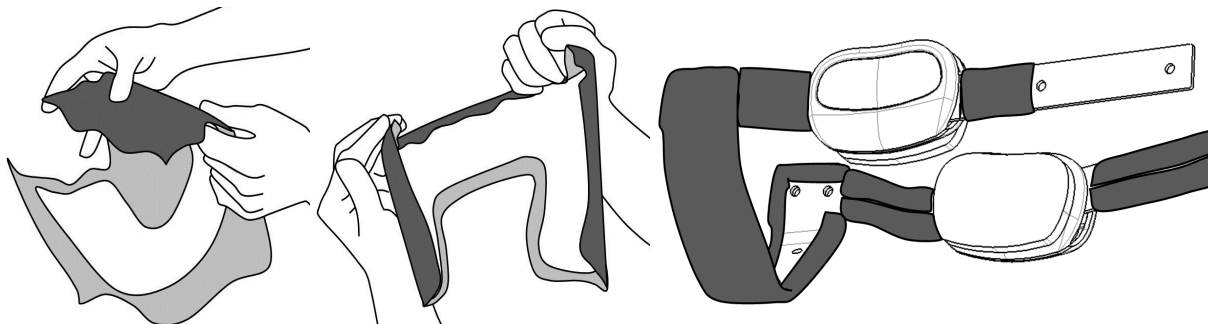

2. Place the micro-Velcro® round pieces inside the AK band. The micro-Velcro® band must be so stuck inside the BK band that it covers the holes and will be riveted later with the strap and the BK loop. Cut this micro-Velcro® band in two parts if needed (see fig. 1). Before sticking check that the surfaces are clean and degreased. Optimal sticking is achieved after a 48 hours setting time.
3. Use the tubular rivets with black cap (red arrows on fig.1) to assemble the thigh band and the uprights. **Do not forget to rivet the thigh strap and its loop in the same operation.** Popliteal and tibial straps and their respective loops must also be fixed using tubular rivets with black cap.
4. **Cut the 3D fabric** according to the thigh band outline + 5 mm for turn in.
5. Lining (3D fabric for the thigh and copolymer strip for the BK part) can be placed inside the orthosis.

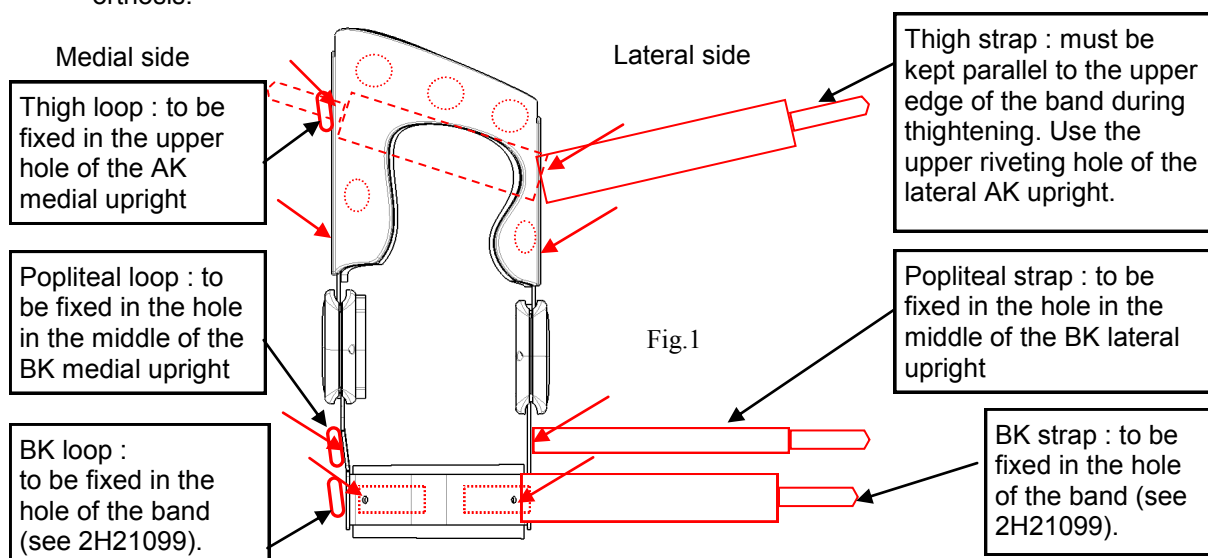2H200-G / 2H200-D Assembling instructions

When cutting the straps to the appropriate length, take care to make a stopping stitch on each edge of the seam. (fig.2).

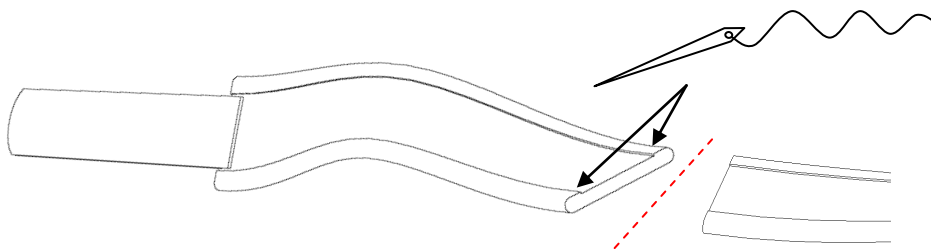
Fig.2

The rigid corners of the strap on each side of the velcro band can cause discomfort for the patient. It may be necessary to cut them. Make sure the seams have not been damaged by cutting (fig.3).

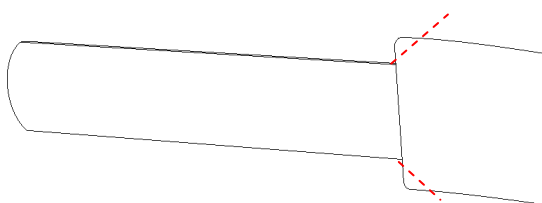
Fig.3

To improve the support provided by the orthosis, add a pad on the thigh and/or tibial strap (cut the pad in the supplied silicone foam strip). (fig.4).

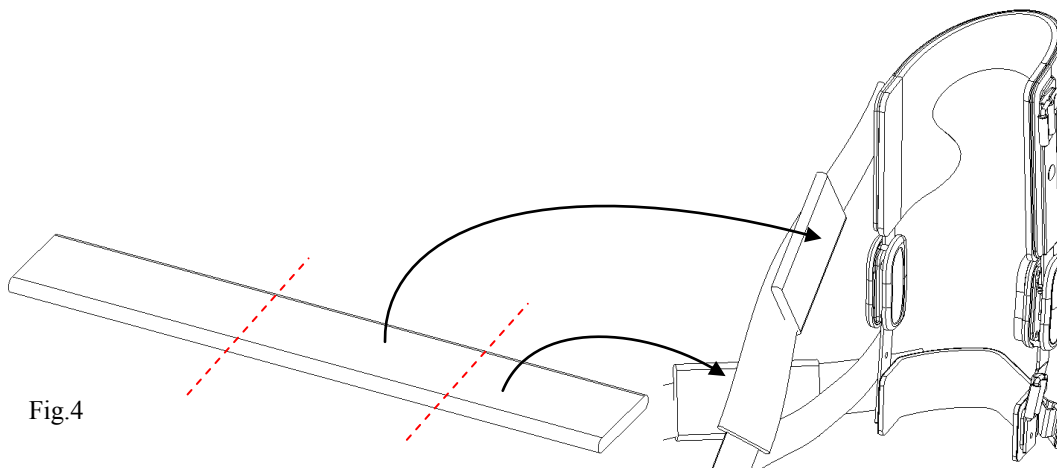
Fig.4

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