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INSTRUCTIONS FOR 1K60 AND 1K66


I. USE

The adapters **1K60** and **1K66** are designed exclusively for lower limb prosthesis. They are appropriate for patients weighing (carried load included) less than 125 kg.

II. CONNECTIONS


The **1K60** and **1K66** adapters can be connected through their pyramid **2** to any « male pyramid » for **1K66** or « female pyramid » for **1K60** ; Through their plate **1** they should be connected to the following components only :

- the **1K61** and **1K62** adapters for sport blade (see assembling instructions §III and rotation adjustment §V).
- the **1K90** junction cup, using the **1K58** kit (not included) (see assembling instructions §IV and rotation adjustment §V).
- the **1P650** Easy Ride sport knee (see rotation adjustment §V).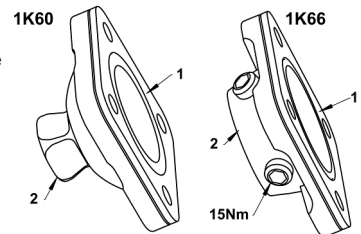


III. HOW TO ASSEMBLE 1K61 AND 1K62

To assemble the **1K60** and **1K66** adapters to the **1K61** and **1K62** components, follow the order of the components indicated on the opposite diagramm (**W** = screws that come with the **1K61** and **1K62** components). Tighten screws **W** up to the tightening torque given in the instruction sheet for **1K61** and **1K62** components.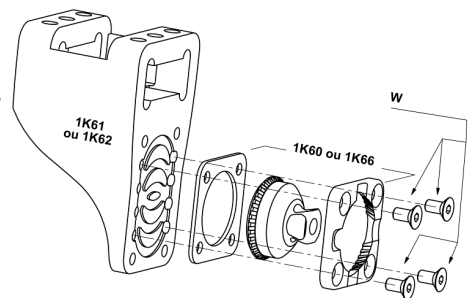


IV. HOW TO ASSEMBLE 1K90

The **1K58** wedging kit (to be ordered separately) must be used to assemble the **1K60** and **1K66** to the **1K90** component : the 16 mm long countersunk screws (**V**) included in this kit must necessarily be used. During assembling, follow the order of the components indicated on the opposite diagramm (**X** = wedge included in the **1K58** kit and **V** = screws included in the **1K58** kit). Tightening torque for the screws : 6 Nm.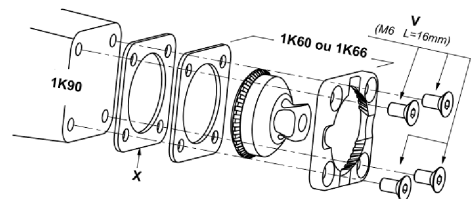


V. ROTATION ADJUSTMENT


a) Undo the 4 screws **W** (2 turns and a half).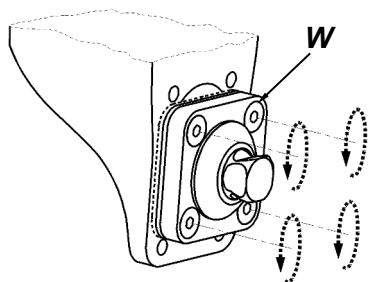


b) Move plate **P** backwards while maintaining pyramid **H** and wedge **T**** against the socket attachment device.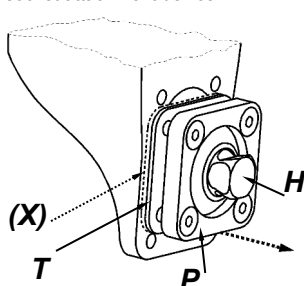

c) Turn pyramid **H** up to the desired position (1 pitch = 6°).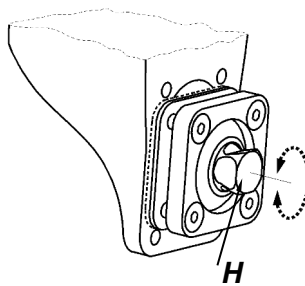

d) Tighten the 4 screws **W** to the recommended tightening torque and check that wedge **T**** is centered on pyramid **H**.

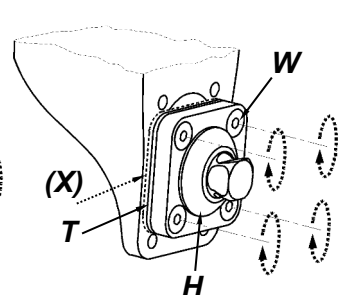(* and wedge X if needed)

Note : The above diagrams show the rotation adjustments for 1K60. The rotation adjustments for 1K66 and 1P650 knee are made in the same way.

V. RECYCLING

The **1K60** and **1K66** adapters include parts in titanium that should be recycled in compliance with the law in force.

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