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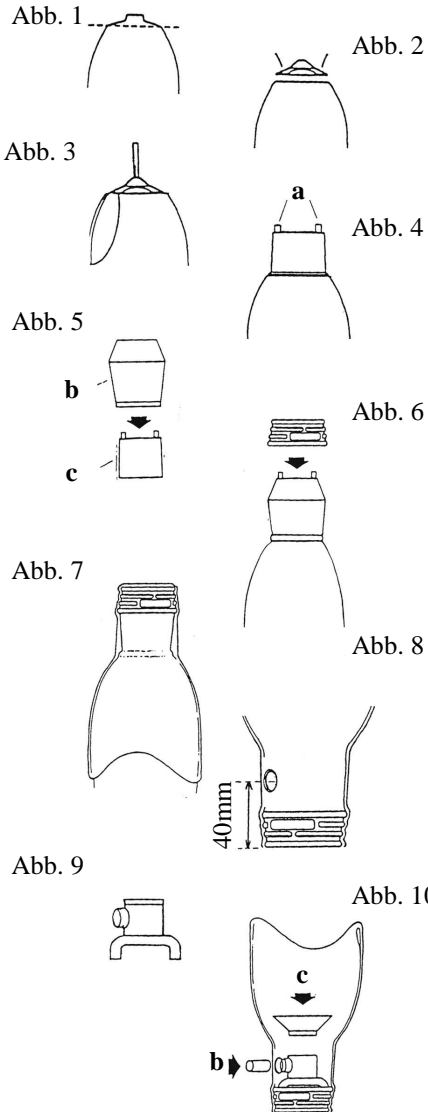
**QUICK DISTAL FASTENING DEVICE
1 S 137-1 SMALL SIZE KIT FOR 1K90 CUP**

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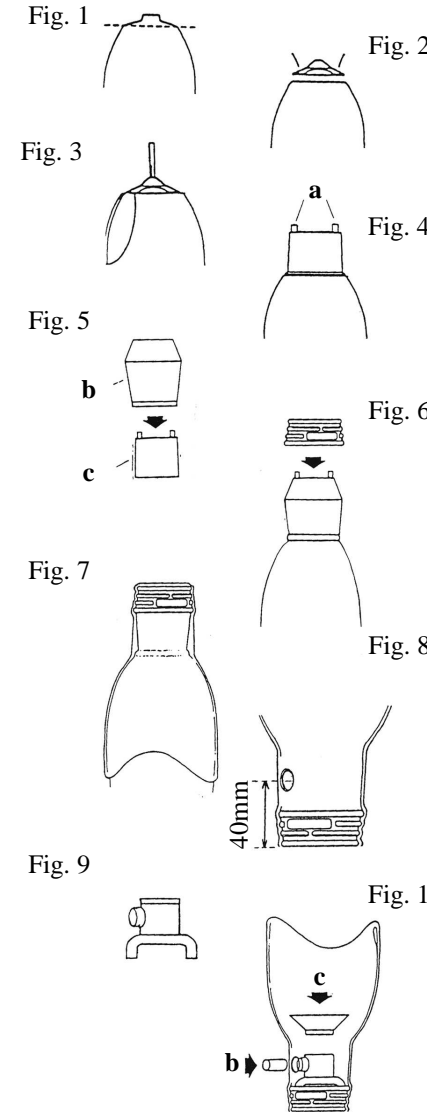
I - 1S137-1 KIT INCLUDING :

- a quick distal fastening device
- a guidance cone - small size
- a linking device 1S108

II - ASSEMBLING :

- 1) Smooth distal part of the positive (fig. 1) and fasten the 1S112 tip with 2 little nails (fig. 2), then shape again its initial contour using plaster. Smooth the assembly.
- 2) Screw on the direction rod in 1S112 (fig. 3) and place the 1S129 tool (fig. 4).
CAUTION ! Take care of the place of the metal centring pins a. They will maintain the 1K90 cup according to the selected push button position (draw a mark).
- 3) Slide the silicone band b on the 1S129 tool after isolation with release agent (fig.5). Take care of the mountig direction for this silicone band.
- 4) On 1S129 tool place the 1K90 cup that is centered by the 2 metal centring pins (fig. 6). Then stick the self-adhesive protecting chip on the bottom of this cup.
- 5) Laminate or thermoform the socket as usually (fig. 7).
- 6) Remove the socket from the plaster and dismantle the forming tool and the tip. In the socket, at 40 mm from the lower extremity of the 1K90 cup, bore a \varnothing 24 mm hole in front of the push button (fig.8) in order to be able to remove it. Screw out the push button.
- 7) Screw the quick fastening device 1S117 onto the 1S108 device with the supplied screws (fig. 9).
- 8) Mount the assembly inside the socket and fasten it to the 1K90 cup (supplied screws).
- 9) Place again the push button b and the guidance cone c (fig. 10).

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